



Safe Electrodes™

302132-200

EEG Gold Cup, 10mm/.40 48" Black
(10/Pkg)



www.cadwell.com



Cadwell

909 N. Kellogg Street
Kennewick, WA 99336

Phones: 1 (800) 245-3001
1 (509) 735-6481

www.cadwell.com



CE partner4U BV
3951 DB; 13. NL



www.cadwell.com/electrodecare

US, CA: R_x



Prepare

- Disconnect items from equipment prior to cleaning or disinfection.



After each use

- Clean with soft brush, water and mild detergent.
- Disinfect with quaternary ammonium or alcohol solution such as Envirocide or Lysol I.C.



Caution

- Inspect before use for damage and excessive wear
- Discard using appropriate procedure
- Follow your facilities approved infection control procedures.



Disclaimer

- Cadwell is not responsible for injury, infection, or other damage resulting from improper storage, preparation, use, or disposal of electrodes.

PN: 302132-622 rev0

www.cadwell.com/electrodecare