



Technomed Europe

Medical Accessories


DIRECTIONS FOR USE

Disposable Needle Electrodes

Applicable for:

- Disposable Concentric Needle Electrodes
- Disposable Monopolar Needle Electrodes
- Disposable Subdermal Needle Electrodes
- Disposable Hypodermic Needle Electrodes

Safe removal of protection sheath: twist and pull.

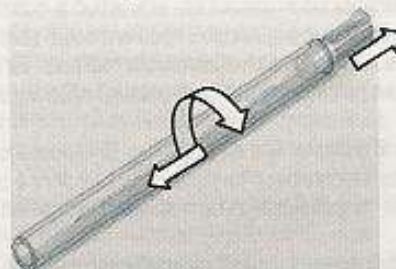

Technomed Europe
Amerikalaan 71
6199 AE Maastricht-Airport
The Netherlands
Tel.: +31 43 408 68 68
E-mail: info@technomed.nl
www.technomed-europe.com

CE
0344



USA
Rx ONLY

STERILE EO

English

DISPOSABLE NEEDLE ELECTRODES

Directions for use

When selecting a disposable needle electrode, it is important to use one that is of suitable length and diameter for the investigation being undertaken.

CAUTION!

- The disposable needle electrodes are sterile unless packaging is damaged or opened or use before date is passed.
- When introducing a disposable needle electrode into a patient it is recommended that it is not inserted up to the hub.
- If the disposable needle electrode bends before, during, or after insertion, do not straighten or re-insert. Bent disposable needle electrodes should be discarded and replaced with a new disposable needle electrode.
- Always discard used disposable needle electrodes in a properly marked medical biohazard sharps container.

WARNING!

These disposable needle electrodes are not intended for use as stimulating electrodes¹ or as patient ground electrodes. If used as recording electrodes, the possible loss of electrocautery return path may result in high current density passing through these needles with possible tissue heating and damage resulting.

For single patient use only. Do not reuse or re-sterilize. Cleaning and re-sterilization of the device can affect the safety, performance and effectiveness of the device and expose patients and users to unnecessary risk.

^{*)} The Disposable Hypodermic Needle Electrode can be used for muscle stimulation, recording EMG and drug delivery.