GAS-ISE Metabolites - Level 1

EN Eurotrol GAS-ISE Metabolites - Level 1

Intended Purpose

Eurotrol GAS-ISE Metabolites - Level 1 is an assayed blood gas, electrolyte and metabolite reference material, to verify the precision and accuracy of the epoeb® Blood Analysis System. Eurotrol GAS-ISE Metabolites - Level 1 was designed to test the following analytes: pH, pO₂, pCO₂, Na⁺, K⁺, Ca⁺⁺, Mg⁺⁺, Cl⁻, Glucose, Lactate, Urea*, Creatinine* and tCO₂. Eurotrol GAS-ISE Metabolites - Level 1 is also developed to verify calibration, operating temperature and performance characteristics.

*Cl-, Urea, Creatinine and Mg++ are not available in Canada

IVD Medical Device

Eurotrol GAS-ISE Metabolites - Level 1 complies with the IVD Directive 98/79/EC for Medical Devices and carries the CE mark.

Eurotrol GAS-ISE Metabolites - Level 1 complies with the following US codes of Federal Regulations (CFR): 42 CFR part 72 and 21 CFR parts 606, 640 and 820.

Eurotrol GAS-ISE Metabolites - Level 1 is for in vitro diagnostic use only.

Summary

Eurotrol GAS-ISE Metabolites - Level 1 is an assayed blood gas, electrolyte and metabolite reference material, to verify the precision and accuracy of the epoeb® Blood Analysis System. Each ampule contains 2.5 mL solution.

Reagents

Eurotrol GAS-ISE Metabolites - Level 1 controls are prepared using salts in a physiologically buffered matrix. Tonometry with predetermined levels of oxygen and carbon dioxide balanced with nitrogen and different salt concentrations provide a distinct level for each parameter; simulating clinically significant ranges of acid-base status, respiratory function, renal function, glucose and lactate concentration. Eurotrol GAS-ISE Metabolites - Level 1 controls contain no preservatives, viscosity adjusters or other additives that might adversely effect electrode measurements.

Storage and Stability

Eurotrol GAS-ISE Metabolites - Level 1 can be stored at a temperature of 2-8 °C (35-46 °F). Stored unopened at this temperature it is guaranteed stable as indicated until the expiry date on the ampules and ampule box. Eurotrol GAS-ISE Metabolites - Level 1 is stable for 1 week at room temperature (unopened ampules). Do not refrigerate the product after it has been brought to room temperature! After opening the ampule, Eurotrol GAS-ISE Metabolites - Level 1 is stable for 30 seconds.

Procedures

- Remove the appropriate amount of ampules from box and tray. Equilibrate the ampule(s) for a minimum of 1 hour to a temperature as close to 22 °C (71.6 °F) as possible.
- Immediately before use, shake the ampule vigorously for at least fifteen seconds to re-equilibrate the gases with the solution. When shaking, the ampule should be held between the thumb and index finger.
- Swift the ampule gently to return the solution to the bottom of the ampule. Allow any bubbles to rise before opening the ampule.
- Protect fingers with gauze, tissue or gloves.
- Hold ampule with the colored dot facing up. Snap off the neck of the ampule in the opposite direction of the colored dot. Please read the OPC instruction leaflet for more details.**
- Within 30 seconds, follow the sampling procedure according to the instructions in the operating manual of your epic Blood Analysis System.
- Please refer to local guidelines for recommended frequency of use.

Precautions

- For in vitro diagnostic use only.
- These products should not be disposed of in general waste. Consult local environmental authorities for proper disposal.
- Eurotrol GAS-ISE Metabolites - Level 1 is not to be used as a calibrator.

Expected Results
Values are obtained by measurements on epic® Blood Analysis System using BGEM test cards. The assigned values are available from <http://www.epocal.com>, tabulated for each fluid lot number and sensor configuration. Eurotrol GAS-ISE Metabolites - Level 1 is intended for professional use in the verification of the instrument performance.

Phase Note

- The values in the table are applicable only to the assigned fluid lot number, the applicable epic® Blood Analysis System sensor configuration and at sea level.
- Incorrect sampling, storage, or other mishandling may cause the readings to deviate from the target values.
- The pO₂ values of Eurotrol GAS-ISE Metabolites - Level 1 vary inversely with temperature changes. For each °C change in temperatures between 18 °C and 26 °C (between 64.5 °F and 79 °F), the pO₂ changes 1.5% in the low level, 1.1% in the normal and 0.5% in the high level. To obtain a high degree of correlation with the values in the table, the ampules should be equilibrated as close to 22 °C (71.6 °F) as possible.
- Once the ampule is opened, room temperature has no significant effect on pH values.
- Eurotrol GAS-ISE Metabolites - Level 1 is sensitive to most of the instrument related factors that can cause unexpected analytical deviations.
- Eurotrol GAS-ISE Metabolites - Level 1 controls do not contain protein and therefore may not detect malfunctions sensitive to these components.
- Eurotrol GAS-ISE Metabolites - Level 1 is very sensitive to room air contamination. When sampling the fluid within 30 seconds, the error is less than 2% over the entire range in all levels*. The effect of room air on pH is negligible.

*Numbers are based on open ampule testing.

This product has been manufactured according to Eurotrol specifications.

For further information please contact:

Eurotrol B.V., The Netherlands, Phone +31 318 695 777, Fax +31 318 695 770
E-mail office@eurotrol.com, www.eurotrol.com
Eurotrol Inc., USA, Toll free phone 1-866-234-5754, Main phone 1-978-598-3779, Fax 1-978-598-3780, E-mail officeUSA@eurotrol.com, www.eurotrol.com

Eurotrol B.V., Keplerlaan 20, 6716 BS Ede, The Netherlands

Symbols used	
	Attention, see instructions for use
	Use by
	Temperature limitation
	In Vitro Diagnostic Medical Device
	Manufacturer
	CE Mark

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Eurotrol B.V., Niederlande, Tel. +31 318 695 777, Fax +31 318 695 770
Eurotrol B.V., Keplerlaan 20, 6716 BS Ede, Nederland