




Preparing the Inflation Syringe and Extension Line

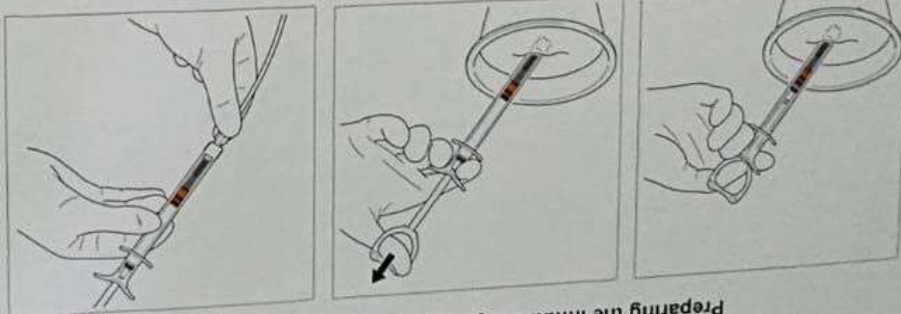


1. Begin with the inflation syringe plunger all the way in (Figure 1).

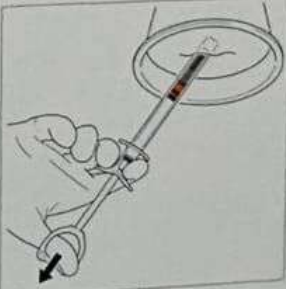


2. Then, submerge tip in sterile saline solution.

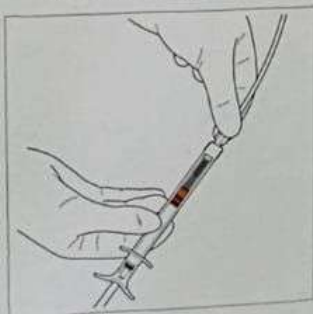

3. Fill inflation syringe by slowly drawing plunger back to second click position (all the way out) (Figure 3).



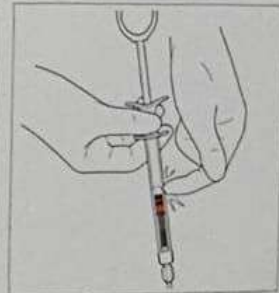


5. Point the syringe tip toward the ceiling. Tap the inflation syringe until a large bubble is visible beneath the orange piston.

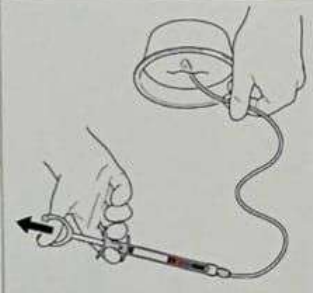

6. While still pointing the syringe tip toward the ceiling, push the plunger all the way in (Figure 1) to purge all air and fluid from the syringe.

Note the 3 referenced inflation syringe plunger positions:

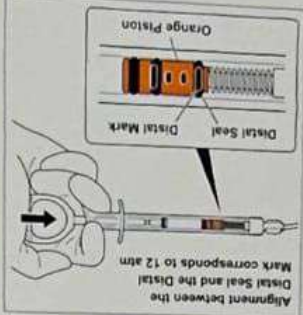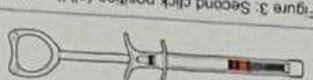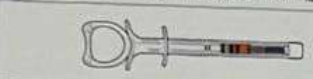

7. Submerge the free end of the extension line in sterile saline solution. Slowly draw plunger back to the first click position (Figure 2) to fill the syringe.



8. Connect the free end of the extension line to the Xpress inflation luer.

Note: Inspect the syringe barrel to ensure there is minimal air in the system. If excessive air remains in the system, repeat the prepping process.

9. Remove and discard the balloon protector.

10. Perform a test inflation of the system by depressing the plunger rod until the distal black seal on the orange piston is aligned with the distal black mark of the inflation syringe. If the seal and black mark do not align, disconnect the inflation syringe and extension line, and repeat the prepping process.

Please see enclosed INSTRUCTIONS FOR USE for complete instructions for use of the Xpress™ ENT Dilation System.

Patent: www.ent.stryker.com/patents

stryker

Xpress® LoProfile

ENT Dilation System

stryker
SISTEMA DE DILATACION XPRESS
ENT
Importado por: Stryker Colombia S.A.S
Calle 115 No. 7-13, Of. 1001
Bogotá, Colombia
Registro Sanitario: No. 144154-2017-001-0017000

XprESS LoProfile
ENT Dilation System

6x20mm

APROBADO
s  **Medinistros**
Instituto Colombiano de Normas Técnicas y Certificación

REF LPLF-106

LOT A26856

MODEL MSB&LLF

 2025-03-08

CE
2107

STERILE EO

Rx Only



Do Not Reuse

Quantity



Consult instructions for Use

Not Made With
Natural Rubber Latex



Type B1 Applied Part


Stryker Medical Inc.
3610 Holly Lane North, Suite 40
Plymouth, MI 48170 USA
+1 888.820.7815
www.entstryker.com
Stryker Medical, Inc. a Stryker Company

EC REP
ACON S.R.L. Limited
South County Business Park
Levenshall, Dublin 18
D18 XN63, Ireland



051108875003014
170263034
100420008
21030723081047190