

STERILE

EO



CE 0123

Endotracheal tube

TYPE: Cuffed

I.D.: 4.0 mm

O.D.: 5.5 mm

Importado por Well Medicine Products S.A.
C 69B # 77- 41, Barrio las Ferias

 well

ENDOTRACHEAL TUBE

STERILE EO



CE 0123

Warning:
• Single use only.
• Storage if package is undamaged or undamaged.
• Do not resterilize.
• Do not expose to temperatures above 49°C.

FABRICANTE: WELLMEDICAL INTERNATIONAL CO., LTD
ADD. NO. 412-B SHICHANG ROAD, 264000 WEIHAI, SHANDONG
PROVINCE, PEOPLE'S REPUBLIC OF CHINA
TEL: 862631823152

Importado por Well Medicine Products S.A.
C 69B # 77- 41, Barrio las Ferias
Bogotá-Columbia
TEL: (051) 2223239
Registro Invención N° INV/IMA-2017CM-0516353



Directions:

- Prior to intubation, deflate the cuff completely.
- After intubation, inflate the cuff using the minimum volume of air required to provide an effective seal.
- Immediately after cuff inflation, auscultate both lung fields. If breath sounds diminished over one lung field or absent over one or both fields, adjust the tube as required.
- Endotracheal tube placement should be confirmed by viewing the position of the tube tip with a chest radiograph.

