XS

TP - ECH

MICRO $\oplus$ FLEX[®] SUPRENO[™] EC

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NITRILE

EXAMINATION GLOVES
POWDER-FREE
TEXTURED FINGERS
EXTENDED CUFF
SINGLE USE ONLY
AMBIDEXTRORS

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* WARNING: DO NOT USE WITH CARMUSTINE AND THIOTEPA.

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145TM 06376-051

Tested for use with chemotherapy drugs: Specific chemical resistance data available upon request. Gloves used for protection against chemotherapy drugs must be selected specifically for the type of chemicals used. Review material safety sheets for the chemicals being used to determine the required protection level.

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AHC_SUE00432A

THIS GLOVE IS FOR SINGLE USE ONLY! THIS GLOVE MEETS THE SINGLE-USE EMERGENCY MEDICAL EXAMINATION GLOVE REQUIREMENTS OF NFPA 1999, STANDARD ON PROTECTIVE CLOTHING AND ENSEMBLES FOR EMERGENCY MEDICAL OPERATIONS, 2018 EDITION. DO NOT REMOVE THIS LABEL!

PROTECTIVE SINGLE-USE EXAMINATION GLOVES FOR EMERGENCY MEDICAL OPERATIONS IN ACCORDANCE WITH NFPA 1999-2018 (35VF)

Proper use consistent with NFPA 1500, Standard on Fire Department Occupational Safety and Health Program; NFPA 1581, Standard on Fire Department Infectious Control Program; 29 CFR 1910.132, "General Requirements of Subpart I, Personal Protective Equipment," and 29 CFR 1910.1030, "Protecting Health Care Workers from Occupational Exposure to Blood-Borne Pathogens." Most performance properties cannot be tested by the user in the field.

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CAUTION: Components used in making these gloves may cause allergic reactions in some people. Safe use of this glove by or on nitrile sensitized individuals has not been established. If allergic reactions occur, discontinue use immediately and consult a physician. Follow your institution's policy for use. Instructions for use: For single and transient use only. Wear and dispose of these gloves in accordance with industry standards and industry best practices. Inspection of gloves for defects by the user is recommended during donning and use. Immediately discontinue use if damaged. **Disclaimer of Warranties and Limitations of Remedies:** To the maximum extent permitted by law, seller disclaims all express and implied warranties, including without limitation, all implied warranties of merchantability and fitness for a particular purpose or use. There are no warranties which extend beyond the description on the face hereof. It is understood and agreed that seller's liability, whether in contract or otherwise, shall not exceed the amount of the purchase price paid by buyer and under no circumstances shall seller be liable for special, incidental, consequential or indirect damages for personal injury, loss of business, business interruption, or any other loss arising from the use or non-use of this product. The price for these goods is a consideration in limiting seller's liability. No action arising from the use or non-use of these gloves may be brought more than one year after the cause of action has accrued. These gloves are purchased "as is". User assumes all risk of possible allergic sensitization or reactions. **Safety Considerations:** According to the "NIOSH ALERT", use of non-natural rubber latex gloves (non-latex gloves) is an effective method of preventing allergic reactions to natural rubber latex in the workplace. NIOSH has recommended that non-latex gloves, such as nitrile gloves, be used to protect workers from latex exposure and allergy in the workplace. For additional information about the "NIOSH ALERT", contact the National Institute for Occupational Safety and Health (NIOSH) directly at (800) 358-4674.

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NITRILE

EXAMINATION GLOVES
POWDER-FREE
TEXTURED FINGERS
EXTENDED CUFF
SINGLE USE ONLY
AMBIDEXTROUS

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Ansell, ® and ™ are trademarks owned by Ansell Limited or one of its affiliates. © 2021 Ansell Limited. All Rights Reserved.