

MEDI-GRIP® Latex Standard

CHEMOTHERAPY PERMEATION REPORT

CHEMOTHERAPY DRUG	MINIMUM BREAKTHROUGH DETECTION TIME (MINUTES)	CHEMOTHERAPY DRUG	MINIMUM BREAKTHROUGH DETECTION TIME (MINUTES)
	ASTM D6978		ASTM D6978
Carboplatin/Paraplatin (10.0 mg/ml)	>240	Irinotecan (20.0 mg/ml)	>240
Carmustine (3.3 mg/ml)	11.8	Methotrexate (25.0 mg/ml)	>240
Cisplatin (1.0 mg/ml)	>240	Mitoxantrone (2.0 mg/ml)	>240
Cyclophosphamide (20.0 mg/ml)	>240	Oxaliplatin (2.0 mg/ml)	>240
Docetaxel (10.0 mg/ml)	>240	Paclitaxel (6.0 mg/ml)	>240
Doxorubicin Hydrochloride (2.0 mg/ml)	>240	Rituximab (10.0 mg/ml)	>240
Etoposide (20.0 mg/ml)	>240	ThioTEPA (10.0 mg/ml)	13.4
Fluorouracil (50.0 mg/ml)	>240	Vincristine Sulfate (1.0 mg/ml)	>240
Gemcitabine (38.0 mg/ml)	>240	Vinorelbine (10.0 mg/ml)	>240
Ifosfamide (50.0 mg/ml)	>240		

RATING SYSTEM

RECOMMENDED

The results for this specific chemotherapy drug suggest that the glove would provide an adequate barrier for use in most applications. Breakthrough occurs in >60 minutes.

CAUTION

The results require additional consideration to determine suitability for use. Breakthrough occurs in >10 to 60 minutes.

NOT RECOMMENDED

Not recommended for use. Breakthrough occurs in ≤10 minutes.

IMPORTANT: ASTM D6978 testing performed by a third-party accredited laboratory. Stated breakthrough times were determined under laboratory conditions that may not reflect actual usage. Variation in the environment or a mix of chemotherapy drugs used may impact breakthrough times. Users should test the suitability of this product against their specific chemotherapy drugs and environment.



MEDI-GRIP®

Trusted around the world to deliver the everyday protection that surgeons, O.R. nurses, and surgical technicians can depend on to perform their daily tasks safely and comfortably.

Contact your Ansell representative for ordering or more information.

North America

US Tel: 800 952 9916
CA Tel: 1-844-494-7854
Email: insidesalesus@ansell.com

Central & South America

Tel: +52(442) 296 20 50
Email: cslac@ansell.com
Brazil
Tel: +55-11-3356-3100
Email: luvas.medicas@ansell.com

Europe, Middle East & Africa

Tel: +32 (0) 2 528 74 00
Email: info@ansell.eu

Asia Pacific

Tel: +603 8310 6688
Email: apac.medical@ansell.com

Australia & New Zealand

Tel: +61 3 9270 7270
Email: protection@ap.ansell.com