

Pediatric Pediatric Pediatric Pediatric Pediatric

HUDSON RCI®


REF 1058
(IPN913892)

Pediatric

**Non-Rebreathing Mask
with safety vent and
7ft. (2.1m) Star Lumen® Tubing**

QTY 1

LOT 00L0000000
2020-01-01
2033-11-22

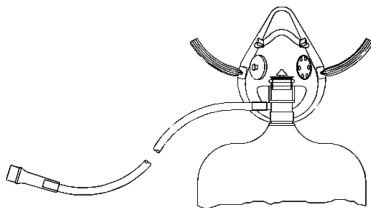

DEHP

Do Not Reuse

Contains DEHP

Rx ONLY

Not made with natural rubber latex.

(01) 1 4026704 64809 9

(17) 331122

(11) 200101

(10) 00L0000000



LBL023250 R00

Pediatric Pediatric Pediatric Pediatric Pediatric Pediatric Pediatric Pediatric Pediatric Pediatric

Pediatric Pediatric Pediatric Pediatric Pediatric

Directions for Use:

NOTE: These directions are general guidelines intended for use by qualified medical personnel.

1. Attach the oxygen supply tubing to the gas source.
2. Set the oxygen to the desired flow.
3. Check for flow through the device. Always verify proper function of the valves:
 - The valve between the mask and the reservoir should rise on inspiration and lower on exhalation.
 - The valve located on external mask surface should open during exhalation.

WARNING: Always check for oxygen flow through oxygen inlet valve before placing the mask on the patient.

4. Place the mask on the patient's face with the elastic strap below the ears and around the neck. Gently pull the strap ends until the mask is secure. Mold the metal strip on the mask to fit the face.

Hudson RCI and Star Lumen are registered trademarks of Teleflex Incorporated or its affiliates.

© Copyright 2011, 2013, 2014, 2017, 2018 Teleflex



**3015 Carrington Mill Blvd
Morrisville, NC 27560 USA
USA: (866) 246 6990**

International: +1 919 544 8000

Product of Mexico, Packaged in Mexico

LBL023250 R00