

Pediatric Pediatric Pediatric Pediatric Pediatric


HUDSON RCI®

REF 1886
(IPN914789)
QTY 1


Pediatric
MICRO MIST® Nebulizer
with Elongated Pediatric
Aerosol Mask and 7 ft Star Lumen® Tubing

LOT 00L0000000
2011-11-11
2099-11-11

Not made with natural rubber latex.

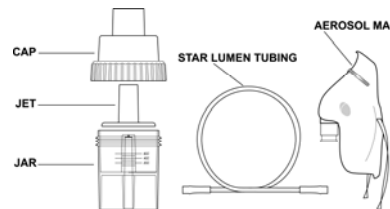

Rx ONLY

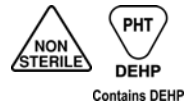

California Prop. 65
WARNING: Cancer and Reproductive Harm.
www.P65Warnings.ca.gov



(01) 1 4026704 67089 2
(17) 991111
(11) 111111
(10) 00L0000000



LBL027573 R00

Pediatric Pediatric Pediatric Pediatric Pediatric

Directions for Use:

NOTE: These directions are general guidelines intended for use by qualified medical personnel.

1. Place prescribed solution into the nebulizer jar.
2. Attach cap to nebulizer jar with a clockwise rotation.
3. Attach the supply tubing between the gas source and the nebulizer.
4. Connect the nebulizer to the mask inlet.
5. Set gas flow to 8 LPM. Check for aerosol mist; if necessary, tap the device until it begins to nebulize.
6. Place the mask on the patient's face with the elastic strap below the ears and around the neck. Gently pull the ends of the strap until the mask is secure. Shape the metal strap on the mask to fit the face.
7. During treatment, periodically tap nebulizer to minimize residual volume.

WARNING: Be sure all connections are secure.

WARNING: Do not attach the outlet of the nebulizer directly to an endotracheal tube, tracheostomy tube or other device that does not offer an alternate pathway for exhalation.

STAR LUMEN, MICRO MIST and Hudson RCI are registered trademarks of Teleflex Incorporated or its affiliates.

© Copyright 2011, 2014, 2016, 2018, 2019 Teleflex



3015 Carrington Mill Blvd
Morrisville, NC 27560 USA
USA: (866) 246 6990
International: +1 919 544 8000
Product of Mexico, Packaged in Mexico

LBL027573 R00