

ADULT

QTY 1

Adult MULTI-VENT® Air Entrainment Mask


LOT 00L0000000



2011-11-11



2099-11-11

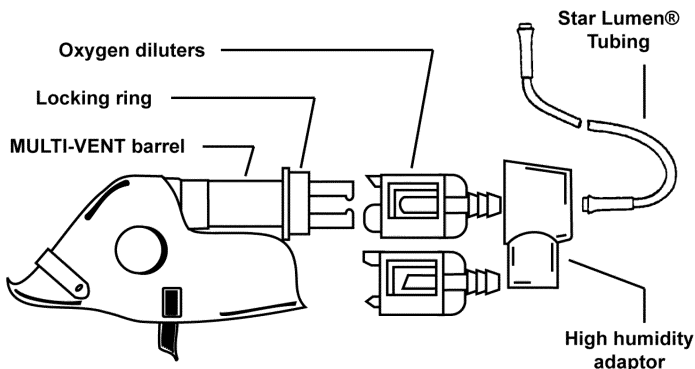

Instructions for Use:

NOTE: These directions are general guidelines intended for use by qualified medical personnel.

1. Select the appropriate oxygen diluter (green-24%, 26%, 28% or 30%; white for 35%, 40% or 50%).
2. Slip the diluter onto the MULTI-VENT barrel.

Not made with natural rubber latex.



Do Not Reuse



Contains DEHP

Rx ONLY

(01) 1 4026704 71293 6

(17) 991111

(11) 111111

(10) 00L0000000



3. Select the prescribed oxygen concentration by setting the indicator on the diluter to the appropriate percentage on the barrel.
4. Firmly slide the locking ring into position over the diluter.
5. If humidification is desired, use the high humidity adaptor. To install, match the grooves on the adaptor with the flanges on the diluter and slide firmly into place. Connect the adaptor to the humidity source with large bore tubing (not supplied).

WARNING: Use only room air with the high humidity adaptor. Use of oxygen will affect the desired concentration.

6. Connect supply tubing to the diluter and to the appropriate oxygen source.
7. Adjust the oxygen flow to the appropriate level (see table below) and check for gas flow through the device.

NOTE: Listed flow rates are recommendations only. Individual circumstances may require flows other than those suggested.

FIO ₂	Air : O ₂ Ratio	Recommended O ₂ Liter Flow	Total Gas Flow (to patient)
24%	25.3 : 1	3 LPM	79 LPM
26%	14.8 : 1	3 LPM	47 LPM
28%	10.3 : 1	6 LPM	68 LPM
30%	7.8 : 1	6 LPM	53 LPM
35%	4.6 : 1	9 LPM	50 LPM
40%	3.2 : 1	12 LPM	50 LPM
50%	1.7 : 1	15 LPM	41 LPM

8. Place the mask on the patient's face, with the elastic strap below the ears and around the neck. Gently pull the strap ends until the mask is secure. Mold the metal strip on the mask to fit the face.

WARNING: Use of a humidifier other than those recommended may create excessive back pressure, causing the relief valve to activate and thereby affecting gas flow to the patient.

MULTI-VENT, Star Lumen and Hudson RCI are registered trademarks of Teleflex Incorporated or its affiliates.

© Copyright 2009, 2013, 2017, 2018, 2020 Teleflex



3015 Carrington Mill Blvd
Morrisville, NC 27560 USA
USA: (866) 246 6990

International: +1 919 544 8000

Product of Mexico, Packaged in Mexico

LBL052358 R00 (2020-11)