


ADULT SELECT-A-VENT™

Air Entrainment Mask Kit with Fixed Diluter

QTY 1

LOT 00L0000000
2011-11-11
2099-11-11

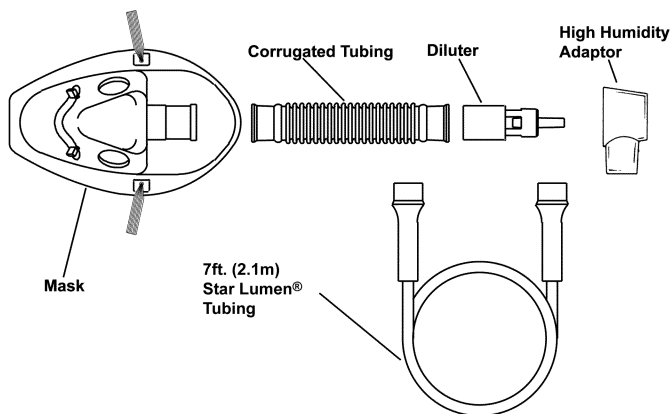

Instructions for Use:

NOTE: These directions are general guidelines intended for use by qualified medical personnel.

1. Attach corrugated tubing onto mask grommet.
2. Select the appropriate oxygen diluter (blue-24%, yellow-28%, white-31%, green-35%, pink-40% or orange-50%).
3. Attach large end of diluter to opposite end of corrugated tubing.

NOTE: Diluter percentages **and** liters per minute are imprinted on diluter.

Not made with natural rubber latex.



Do Not Reuse Contains DEHP

Rx ONLY

(01) 1 4026704 71294 3
(17) 991111
(11) 111111
(10) 00L0000000



4. If desired, press the high humidity adaptor over the distal end of the diluter. Connect corrugated tubing (not supplied) between the adaptor and a humidity source.

WARNING: Use only room air with the high humidity adaptor. Use of oxygen will affect the desired concentration.

5. Connect supply tubing to the diluter and to an appropriate oxygen source.
6. Adjust the oxygen flow to the appropriate level (see table) and check for gas flow through the device. Listed flow rates are recommendations only. Individual circumstances may require flows other than those suggested.

Color	FIO ₂	Recommended O ₂ Liter Flow	Total Gas Flow (to Patient)
Blue	24%	3 LPM	78 LPM
Yellow	28%	6 LPM	66 LPM
White	31%	8 LPM	72 LPM
Green	35%	12 LPM	72 LPM
Pink	40%	15 LPM	60 LPM
Orange	50%	15 LPM	40 LPM

7. Place the mask on the patient's face with the elastic strap below the ears and around the neck. Gently pull the strap ends until the mask is secure. Mold the metal strip on the mask to fit the face.

CAUTION: Use of a humidifier with a low concentration diluter (i.e., 24%, 28%, 31%) may cause the humidifier pressure relief valve to activate and consequently affect gas flow to the patient.

SELECT-A-VENT, Star Lumen and Hudson RCI are trademarks or registered trademarks of Teleflex Incorporated or its affiliates.

© Copyright 2010, 2013, 2017, 2018, 2020 Teleflex



3015 Carrington Mill Blvd
Morrisville, NC 27560 USA
USA: (866) 246 6990
International: +1 919 544 8000
Product of Mexico, Packaged in Mexico

LBL052351 R00 (2020-11)