

Kingsystems **FLEX CONNECTOR** WITH DOUBLE SWIVEL ELBOW

REF	301035	EXP	2021-03-04
LOT	1000508829	EXP	2024-03-04

SUGGESTED PROCEDURE:

1. Check product for occlusions prior to procedure.
2. Attach flex connector to endotracheal tube or face mask.
3. Attach opposite end of flex connector to circuit.
4. Perform procedure. Check to ensure all connections are secure and no leakage occurs.
5. Adjust flex connector to desired position.
6. Discard after single use.

WARNING:
 Do not reuse. This circuit has been designed for one time use only. This circuit may result in either harmful retention of residual gases or dangerously increase its breathing resistance.

CAUTION:

Federal law (U.S.) restricts this device to sale by or on the order of a physician (or properly licensed practitioner).



King Systems
 15011 Merriman Blvd.
 Philadelphia, PA 19154-4233
 Tel: 800-675-5464
 www.kingsystems.com



PKM-43, V04 2019/06

482603 301035
 0171001 2020 210217 202024 10100450829